

Алматы (7273)495-231
Ангарск (3955)60-70-56
Архангельск (8182)63-90-72
Астрахань (8512)99-46-04
Барнаул (3852)73-04-60
Белгород (4722)40-23-64
Благовещенск (4162)22-76-07
Брянск (4832)59-03-52
Владивосток (423)249-28-31
Владикавказ (8672)28-90-48
Владимир (4922) 49-43-18
Волгоград (844)278-03-48
Вологда (8172)26-41-59
Воронеж (473)204-51-73
Екатеринбург (343)384-55-89

Ижевск (3412)26-03-58
Иваново (4932)77-34-06
Иркутск (395)279-98-46
Казань (843)206-01-48
Калининград (4012)72-03-81
Калуга (4842)92-23-67
Кемерово (3842)65-04-62
Киров (8332)68-02-04
Коломна (4966)23-41-49
Кострома (4942)77-07-48
Краснодар (861)203-40-90
Красноярск (391)204-63-61
Курск (4712)77-13-04
Курган (3522)50-90-47
Липецк (4742)52-20-81

Магнитогорск (3519)55-03-13
Москва (495)268-04-70
Мурманск (8152)59-64-93
Набережные Челны (8552)20-53-41
Нижний Новгород (831)429-08-12
Новокузнецк (3843)20-46-81
Ноябрьск (3496)41-32-12
Новосибирск (383)227-86-73
Омск (3812)21-46-40
Орел (4862)44-53-42
Оренбург (3532)37-68-04
Пенза (8412)22-31-16
Петрозаводск (8142)55-98-37
Псков (8112)59-10-37
Пермь (342)205-81-47

Ростов-на-Дону (863)308-18-15
Рязань (4912)46-61-64
Самара (846)206-03-16
Саранск (8342)22-96-24
Санкт-Петербург (812)309-46-40
Саратов (845)249-38-78
Севастополь (8692)22-31-93
Симферополь (3652)67-13-56
Смоленск (4812)29-41-54
Сочи (862)225-72-31
Ставрополь (8652)20-65-13
Сыктывкар (8212)25-95-17
Сургут (3462)77-98-35
Тамбов (4752)50-40-97
Тверь (4822)63-31-35

Тольяти (8482)63-91-07
Томск (3822)98-41-53
Тула (4872)33-79-87
Тюмень (3452)66-21-18
Улан-Удэ (3012)59-97-51
Ульяновск (8422)24-23-59
Уфа (347)229-48-12
Хабаровск (4212)92-98-04
Чебоксары (8352)28-53-07
Челябинск (351)202-03-61
Череповец (8202)49-02-64
Чита (3022)38-34-83
Якутск (4112)23-90-97
Ярославль (4852)69-52-93

Киргизия (996)312-96-26-47

Россия (495)268-04-70

Казахстан (772)734-952-31

<https://boule.nt-rt.ru> || buo@nt-rt.ru

Контроллеры и калибраторы



Ensure performance

One of the most important elements of a complete hematology system is the quality control (QC) material. Our Calibrator and Control products are provided as part of the Boule Total Quality Concept to ensure performance of our hematology analyzers.



Easy handling

Boule control materials use barcode scanning for secure handling and assay sheet registration, minimizing the risk of error when manually entering control reference values. Advanced quality control functions built into Boule analyzers include Levey- Jennings charts, XB-function and QC reports.



Use of Boule Cal

- When a hematology analyzer is first installed in an end-user laboratory.
- After major preventive maintenance or replacement of critical parts.
- After yearly service.
- After major instrument repair.
- When results from control materials indicate an unusual trend or shift of analyzer performance.



Use of Boule Con

- Every day as a routine QC procedure.
- At the start and end of a day's work.
- When the analyzer is restarted after a complete shut-down.
- When changing reagents between lots or within the same manufacturing lot.
- When an unexpected abnormal patient result is reported.



Specifications

Article name	Article description	Article no.
Exigo veterinary hematology system		
Boule Vet Con	1 × 4.5 mL	1504026
Boule Vet Con	6 × 4.5 mL	1504027
Boule Vet Cal	1 × 3 mL	1504028
Medonic M32 and Swelab Alfa Plus 3-part hematology systems		
Normal 16 parameters	1 × 4,5 mL	1504019
Low 16 parameters	1 × 4,5 mL	1504020
High 16 parameters	1 × 4,5 mL	1504021
Low, normal, high 16 parameters	6 × 4,5 mL	1504022
Normal 16 parameters	6 × 4,5 mL	1504043
Low 16 parameters	6 × 4.5 mL	1504176
Boule Cal	1 × 3 mL	1504025
Boule Cal	2 × 3 mL	1504045
Medonic M51 5-part and Swelab Lumi hematology systems		
Boule Con-5Diff A1 Tri	3 × 2 × 3 mL	1504518
Boule Con-5Diff A1 Norm	6 × 3 mL	1504519
Boule Con-5Diff A1 Norm	1 × 3 mL	1504520
Boule Con-5Diff A1 Low	1 × 3 mL	1504521
Boule Con-5Diff A1 High	1 × 3 mL	1504522
Boule Cal-5Diff A1	1 × 3 mL	1504517

Quintus 5-part hematology system

Boule Con-5Diff G2 Tri	3 × 2 × 3 mL	1504376
Boule Con-5Diff G2 High	6 × 3 mL	1504377
Boule Con-5Diff G2 Norm	6 × 3 mL	1504378
Boule Con-5Diff G2 Low	6 × 3 mL	1504379
Boule Con-5Diff G2 High	1 × 3 mL	1504380
Boule Con-5Diff G2 Norm	1 × 3 mL	1504381
Boule Con-5Diff G2 Low	1 × 3 mL	1504382
Boule Cal-5Diff G2	1 × 3 mL	1504375

Алматы (7273)495-231
Ангарск (3955)60-70-56
Архангельск (8182)63-90-72
Астрахань (8512)99-46-04
Барнаул (3852)73-04-60
Белгород (4722)40-23-64
Благовещенск (4162)22-76-07
Брянск (4832)59-03-52
Владивосток (423)249-28-31
Владикавказ (8672)28-90-48
Владимир (4922) 49-43-18
Волгоград (844)278-03-48
Вологда (8172)26-41-59
Воронеж (473)204-51-73
Екатеринбург (343)384-55-89

Ижевск (3412)26-03-58
Иваново (4932)77-34-06
Иркутск (395)279-98-46
Казань (843)206-01-48
Калининград (4012)72-03-81
Калуга (4842)92-23-67
Кемерово (3842)65-04-62
Киров (8332)68-02-04
Коломна (4966)23-41-49
Кострома (4942)77-07-48
Краснодар (861)203-40-90
Красноярск (391)204-63-61
Курск (4712)77-13-04
Курган (3522)50-90-47
Липецк (4742)52-20-81

Киргизия (996)312-96-26-47

Магнитогорск (3519)55-03-13
Москва (495)268-04-70
Мурманск (8152)59-64-93
Набережные Челны (8552)20-53-41
Нижний Новгород (831)429-08-12
Новокузнецк (3843)20-46-81
Ноябрьск (3496)41-32-12
Новосибирск (383)227-86-73
Омск (3812)21-46-40
Орел (4862)44-53-42
Оренбург (3532)37-68-04
Пенза (8412)22-31-16
Петрозаводск (8142)55-98-37
Псков (8112)59-10-37
Пермь (342)205-81-47

Россия (495)268-04-70

Ростов-на-Дону (863)308-18-15
Рязань (4912)46-61-64
Самара (846)206-03-16
Саранск (8342)22-96-24
Санкт-Петербург (812)309-46-40
Саратов (845)249-38-78
Севастополь (8692)22-31-93
Симферополь (3652)67-13-56
Смоленск (4812)29-41-54
Сочи (862)225-72-31
Ставрополь (8652)20-65-13
Сыктывкар (8212)25-95-17
Сургут (3462)77-98-35
Тамбов (4752)50-40-97
Тверь (4822)63-31-35

Казахстан (772)734-952-31

Тольяти (8482)63-91-07
Томск (3822)98-41-53
Тула (4872)33-79-87
Тюмень (3452)66-21-18
Улан-Удэ (3012)59-97-51
Ульяновск (8422)24-23-59
Уфа (347)229-48-12
Хабаровск (4212)92-98-04
Чебоксары (8352)28-53-07
Челябинск (351)202-03-61
Череповец (8202)49-02-64
Чита (3022)38-34-83
Якутск (4112)23-90-97
Ярославль (4852)69-52-93

<https://boule.nt-rt.ru> || buo@nt-rt.ru